Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashi-ward-ku, Kyoto 605-0883,
Japan

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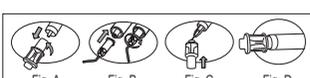
EN Fluoride Releasing Bulk Fill Flowable Restorative Material

BEAUTIFIL-Bulk Flowable is a visible light-cured radiopaque flowable restorative material. It is predominantly designed to be used as a base in Class I and II restorations and can be placed in 4 mm increments. BEAUTIFIL-Bulk Flowable is overlaid with universal or posterior restorative material to replace missing occlusal/facial enamel.

SHADES
Universal(U), Dentin(D)

COMPOSITION
Bis-GMA, UDMA, Bis-MPEPP, TEGDMA, S-PRG filler based on fluoroboroaluminosilicate glass, Polymerization initiator, pigments and others

INDICATIONS
1. Base in Class I and II direct restorations
2. Liner under direct restorative materials
3. Restorations of small posterior cavities (including occlusal restorations)

PRECAUTIONS

- Do not use this product on patients with known allergies to this material and/or methacrylate monomer.
- Operators with known allergies to this material and/or methacrylate monomer must not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator while using this product, immediately discontinue use and seek medical advice.
- Use medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers can penetrate through the gloves very quickly to reach the skin. Therefore, should this material contact even the gloves, immediately remove them and wash hands with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- Cleaning of tooth surface
Thoroughly clean the tooth surface to remove plaque.
- Shade selection
Select the appropriate shade of universal or posterior restorative material using the respective shade guide while the tooth is still in moist.
- Cavity preparation
Prepare the cavity following the conventional method.
- Isolation

- Use a rubber dam or isolate the treatment area from contamination.
- Placement of matrix
Place a matrix band in the case where the cavity involves proximal area. Follow the matrix manufacturing's instructions for use.
- Pulp protection
In the case where the affected area is close to the pulp, cap the pulp with calcium hydroxide etc.
- Adhesive application
Apply dentin adhesive following the manufacturer's instructions for use.
- Delivery
Follow the instructions below to attach the BEAUTIFIL Needle Tip provided.
1) Pull off the shield cap while turning it slowly (Fig.A)
2) Attach the needle tip by turning it clockwise until it is securely attached (Fig.B).
- Placement
Dispense the material directly into the cavity from the bottom to at least 2 mm short of the occlusal cavosurface. In order to prevent the formation of air bubbles, gradually withdraw the tip as cavity is filled. In deeper preparations, place material in 4 mm increments.
- Light-curing
Light-cure each layer (up to 4 mm increment) using Halogen light-curing unit or LED light-curing unit.

Curing time and depth of cure by each light-curing unit (Fig.A)

Shade	Depth of cure	Halogen light-curing unit	LED light-curing unit
Universal	4 mm	20 sec.	10 sec.
Dentin		40 sec.	20 sec.

* Halogen light-curing unit
Light source: Halogen, Wavelength: 400-500 nm, Light intensity: 500 mW/cm² or more
* LED light-curing unit
Light source: Blue LED (with a single emission peak), Wavelength: 440-490 nm, Light intensity: 1000 mW/cm² or more

11. Complete the restoration

Place universal or posterior restorative material directly over the cured BEAUTIFIL-Bulk Flowable. After light-curing, contour, finish/polish the restoration following the conventional method.

NOTE ON USE

- Use a dental light-curing unit having specifications equivalent to those described above. When using the one having performance other than these recommendations, follow the instructions of each unit to light-cure BEAUTIFIL-Bulk Flowable properly.
- Operators must use protective glasses or light shielding plate to avoid looking directly at the curing light.
- In the case where pulp is exposed or the affected area is close to the pulp, cap the pulp with calcium hydroxide etc.
- Do not use eugenol containing materials since they might inhibit polymerization or adhesion of the material.
- Needle tips should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination.
- Store this product in either way of the following.
1) Discard the used needle tip and attach new one. Insert the needle into the small hole on the head of the shield cap (Fig.C)
2) Discard the used needle tip and replace the shield cap securely (Fig.D).
- Carefully handle the shield caps since they are fragile.
- Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.
- When storing these materials in a refrigerator, allow them to reach room temperature before use.
- Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package and container.
(Example: ☞ YYYY-XX → Shelf life: at the end of XX month, YYYY year)

STORAGE

Store in a dark place at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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