for Finishing and Polishing

The OneGloss polisher was developed for finishing and polishing of composite restorations, enamel surfaces and removal of excess cements. To complete the finishing and polishing steps, alter the contact pressure of the OneGloss polisher.

Indications

1. Finishing and polishing of composite restorations.
2. Removal of excess resin or cements on tooth surface without marring the enamel.
3. To remove cement and to polish enamel after orthodontic bracket removal and debanding.
4. Polishing enamel after tooth scaling.
5. To remove stains from tooth surface.




Directions for Use

- Polishing composite
 1. Apply heavy pressure for finishing.
(For gross reduction, recommend using T&F Hybrid points or Super-Snap coarse (Black) disks.)
 2. Apply feather light pressure for polishing.
- Removal of excess resin and cements
 1. Remove excess cements and polish tooth surface using moderate to a feather light touch.
- Polishing enamel and stain removal
 1. Use feather light touch to polish enamel and remove stains.

NOTE: Insert OneGloss Mandrel (with white band) completely into OneGloss polisher. / Use intermittent water spray when applying heavy pressure or polishing enamel. / Disposable

OneGloss Details and Order Information (OneGloss Set PN 0180)

	Shape	Recommended regions to polish	PN
1	Cup	cuspid, incisal, occlusal	0181
2	Midi	occlusal, labial, lingual, buccal	0182
3	IC (inverted Cone)	labial, lingual, buccal, interproximal	0183
4	OneGloss Mandrel CA		0435

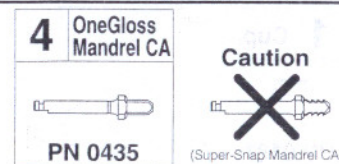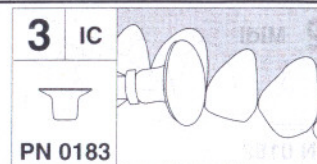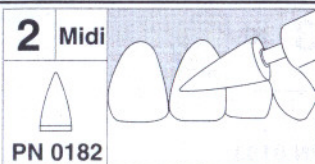OneGloss

Recommended r.p.m.	Maximum r.p.m.	Recommended pressure
3,000-10,000	15,000	0.3 N-for polishing 1.0 N-for finishing

Caution

- Operate each abrasive up to maximum permissible speed only.
- Insert the shank completely into the chuck of handpiece.
- Rotate each abrasive each time prior to use to check smooth run and safety.
- Operate at recommended pressure only.
- Do not use at an acute angle.
- Wear eye protecting glasses when using.
- Use these products for indicated purpose only.

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SHOFU INC., 11 Kamitakamatsu-cho, Fukui, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU INC., Singapore Branch, 215 Henderson Road, #03-01, Singapore 159554
SHOFU INC., Shanghai Rep. Office, No. 900 Yishan Road, Room 1004-1006, Hi-Tech Building, Shanghai 200233, China
SHOFU DENTAL CORPORATION, 1225 Stone Drive, San Marcos, CA 92078-4059, USA
SHOFU DENTAL PRODUCTS LTD., Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU, UK
EU REPRESENTATIVE: SHOFU DENTAL GmbH, Am Brühl 17, 40878 Ratingen, Germany

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SHOFU INC., 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU INC., Singapore Branch, Bldg 215 Henderson Road #03-01, Singapore 159554
SHOFU INC., Shanghai Rep. Office, No. 900 Yishan Road, Room 1004-1006, Hi-Tech Building, Shanghai 200233, China
SHOFU DENTAL CORPORATION, 1225 Stone Drive, San Marcos, CA 92078-4059, USA
SHOFU DENTAL PRODUCTS LTD., Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU, UK
EU REPRESENTATIVE: SHOFU DENTAL GmbH, Am Brühl 17, 40878 Ratingen, Germany